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	Audio Messages	Access Control Settings	Analytics Events	Alarms and Events	Generic Events	<
Analytics ev	ents					
Enable	1					
Port:						
9090						
Security						
Security						
Events allo	wed from:					
Al netv	vork addresses					
O Specifi	ed network addresse	s:				
Δ	ddroop					
	duless					
•						
	Import					

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Windows Security

- < ≡
 - 命 Home
 - Virus & threat protection
 - Account protection
 - P) Firewall & network protection
 - 口 Device security
 - \bigotimes Device performance & health
- 🖧 Family options

\bigcirc Virus & threat protection

Protection for your device against threats.

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Profiles & Options	Value	Explanation
IP Notify Type	Custom Configuration	Predefined Configuration: MarcC Alarm Sunds predefined network messages to the NACC alarm list. Acknowledge Repaired prompts MICC user to confirm the message. If the alarm is not acknowledged within the specified adknowledge tim camera triggers a transmission envir. Select Cattorn Configuration to see the entended configuration.
Destination Address	192,168,10,20,8000	Destination Addresses: Receiver IP address and port. Separate IP address and port using a color. Enter one address per line.
	Parallel send to all	Send Order Send addition to one or more destinations. Sequential and parallel will seed a notification to each destination address. Sond to nee on error will step after the first successful additionation or will try the next address if unsuccess
Data Protocol	Raw TCP/IP	Transfer Protocol: Transfer notification data using these protocol headers.
	-next image-	Separater for Raw TCP/IP. Enter the separater for splitting several notification parts in Raw TCP/IP mode. Enter a unique string.
		CGB Pack: Nonolate CCI path beginning with '/' This parameter allows using <u>variables</u> .
		HTTP Authentication tion rame and password for HTTP authentication separated by colon. Example advisormelism
Data Type	Plain text	Rotification Data Select type of IP notification data.
	("name":"\$(ID.NAM)","event":"\$(FPR.ENO)","eventName":"VM2","ip":"\$(ID.ETHERINET)")	Message: Message to include in Plain dust notification data. When using HTTP protocol this tool is used for QUERY_STRING in GET request. This parameter allows using <u>sortables</u> .

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(c)	c71 mx10-32-204-170 Action Group Details	
General Settings	Value	Explanation
Action Group	IQM VM2 Falling	Name: The name is purely informational,
	Enabled	Arming: Controls this action group: Enabled activate the group. Off descrivate the group. Off descrivate the group. St group armed by signal input. Cs group armed by castom signal as defined in <u>General Event Settings</u> .
	(No time table)	 Time table. Time table for this action profile (<u>Time Tables</u>).
Event Selection	Message: KeplerNurseAssist [Message: Foll_Detected] (Message: Not_In_Ded) (Message: In_Bed) (Message: Situation_Normal)	Event Selection: Select the events which will trigger the actions below. Use [Ctd]-(Lick to select more than one event. Events in parentheses need to be activated first.
Action Details	5	Action Deadtime: Time to wait (0.3600 s) before a new action can take place.
	Simultaneously	Action Chaining: Choose how the status of each subaction influences the execution of all others. Simultaneously, and actions are executed simultaneously. Simultaneously und first success: Simultaneously, but as soon as one action succeeds (i.e. has been completed or the phone's spicide up.) all others are terminiated. Consecutively, All actions are executed in the spocified order. Consecutively, and first success: Consecutive execution, but as soon as one action succeeds, the following action are not executed. Consecutively will first failure. Consecutive execution, but as soon as one action fails, the following action are not executed. Consecutively will first failure. Consecutively consecutively
Actions	Value	Explanation
Action 1 Delete	IP Notify: IQM VM2.	Action Type and Profile: Select the Action Profile to be executed. Action Timeseut or Duration: this action num longer than the time specified (0.3600 s), it is aborted and returns an error; 0 to deactivate. For Image Profile action, this is the duration and no error reterms.

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IP Notify Profile 8 Sma	rtSenseFallDetected Delete	
Profiles & Options	Value	Explanation
IP Notify Type	HTTP/HTTPS	 Predefined Configuration: "MrCC Alarm" sends predefined network messages to the MxCC alarm list. Acknowledge Required prompts the MxCC user to confirm the message. If the alarm is not acknowledged within the specified acknowledge time, the camera triggers a transmission error. Select Custom Configuration to see the extended configuration.
Destination Address	dev-smartsense.ascom.com	Destination Addresses: Receiver IP address and port. Separate IP address and port using a colon. Enter one address per line.
HTTP Method	GET	HTTP Method: Transfer data using one of these HTTP methods.
	/api/thirdparty/alerts/trigger-alert?SensorId=c71DEYM&Ale	ertNar CGI-Path: Absolute CGI path beginning with '/'. This parameter allows using <u>variables</u> .
Security	HTTPS	HTTP Schema: Transfer data using these HTTP schemas.
	Basic authentication	Authentication Method: Method for HTTP Authentication.
		HTTP Authentication: User name and password for HTTP authentication separated by colon. Example: admin:meinsm
Data Type	JSON	Notification Data: Select type of IP notification data.

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/api/thirdparty/clients/status?SensorId={{SensorId}}&SensorTy
atusName={{StatusName}}

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IP Notify Profile 9 ConnectedHe	alth Delete		
Profiles & Options	Value		Explanation
IP Notify Type	HTTP/HTTPS	•	Predefined Configuration: "MoCCAIm" responses predefined resultance messages to the MoCC atarm list. Acknowledge iteruined prompts MoCC sure to confirm the message. If the alarm is not acknowledged within the specified acknowledge tim the camera triggers a transmission error. Select Castom Configuration to see the extended configuration.
Destination Address	api.demo.connected-health.nl:3000		Destination Addresses: Receiver IP address and port. Separate IP address and port using a colon. Enter orea address per line.
	Parallel send to all	٠	Send Order: Send notification to one or more destinations. Sequential and parallel will send a notification to each destination address. Send to net one error will stop after the first successful notification or will by the next address if unsuccess
HTTP Method	POST	٠	HTTP Method: Transfer data using one of these HTTP methods.
	/keples/event		CGI-Puth: Absolute CGI path beginning with 'Y. This parameter allows using <u>variables</u> .
Security	HTTP/1.1	۰	HTTP Schema: Transfer data using these HTTP schemas.
	Basic authentication	٠	Authentication Method: Method for HTTP Authentication.
		۵	HTTP Authentication: User name and password for HTTP authentication separated by colon. Example: admin.meinsm
Data Type	JSON	٠	Notification Data: Select type of IP notification data.
	{ "uuid": \$(MSG.LOCAL.KeplerNurseAssist.notification.uuid), "timestamp': \$(MSG.LOCAL.KeplerNurseAssist.notification.timestamp), "type": \$(MSG.LOCAL.KeplerNurseAssist.notification.notificationType), "stream": "\$(D.SAM)", "message": \$(MSG.LOCAL.KeplerNurseAssist.notification.notificationType) }		Message: Message to include in Plain text notification data. When using HTTP protocol this text is used for QUERY_STRING in GET request. This parameter allows using <u>undables</u> .
Send Port	0	0	Port Number: Send a message from this camera port I0 for automatic).

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General Settings	Value	Explanation
Action Group	FallDetected	Name: The name is purely informational.
	Enabled	Arming: Controls this action group: Endéded. activate the group. Off deactivate the group. St group armed by group. St group armed by custom signal as defined in <u>General Event Settings</u> .
	(No time table)	Time Table: Time Table: Time table for this action profile (<u>Time Tables</u>).
Event Selection	()mage Analysis: AS) ()mage Analysis: VM) ()mage Analysis: VM2) Message: NotinBed Message: FallDetected	Event Selection: Select the events which will trigger the actions below. Use [Crtf)-Click to select more than one event. Events in parentheses need to be <u>activated</u> End.
Action Details	5	Action Deadtime: Time to wait (05600 s) before a new action can take place.
	Simultaneously	Action Chaining: Choose how the status of each subaction influences the execution of all others. Simultaneously. All actions are executed simultaneously. Simultaneously unoil first success: Simultaneous execution, but as soon as one action succeeds [i.e. has been completed or the phone is picked up, all others are terminated. Consecutively: All actions are executed in the specified order. Consecutively unoil first success: Consecutive execution, but as soon as one action succeeds, the following actions are not executed. Consecutively: unoil first failure: Consecutive execution, but as soon as one action fails, the following actions are not executed.
Actions	Value	Explanation
Action 1 O Delete	IP Notify: ConnectedHealth	Action Type and Profile: Select the Action Profile to be executed. Action Timeout or Duration: If this action runs longer than the time specified [03600 s], it is aborted and returns an error; 0 to deactivate. For linear before action runs in the time decrete and exercent runs

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Θ	c71 Mobotix-c71-KVEC II	Notify Profiles	© () ⊡ (
IP Notify Profile 10 Si	kyresponseFallDetected Delete		e
Profiles & Options	Value	Explanation	
IP Notify Type	HTTP/HTTPS	 Predefined Configuration: ¹McCC Alarm² sends predefine list. Acknowledge Required pri message. If the alarm is not acl acknowledge time, the camera Select Custom Configuration to 	d network messages to the MACC alarm ampts the MACC user to confirm the knowledged within the specified triggers a transmission error. a see the extended configuration.
Destination Address	sandbox.skyresponse.com	Destination Addresses: Receiver IP address and port. Separate IP address and port u Enter one address per line.	sing a colon.
	Parallel send to all	Send Order: Send notification to one or mo Sequential and parallel will sen address. Send to next on enrorwill stop try the next address if unsucce	re destinations. of a notification to each destination after the first successful notification or v ssful.
HTTP Method	POST	HTTP Nethod: Transfer data using one of thes	e HTTP methods.
	/api/alarms	CGI-Path: Absolute CGI path beginning w This parameter allows using yo	ith '/'. riables-
Security	HTTPS	HTTP Schema: Transfer data using these HTTI	schemas.
	None	Authentication Method: Method for HTTP Authentication	m,
		HTTP Authentication: User name and password for H Example: admin:meinsm	TTP authentication separated by colon.
Data Type	JSON	Notification Data: Select type of IP notification data	sta.
	{ "authentication": { "identifier": " "secret": "39yAaWeyfwPJWMjg", "product": "KPLR"	Message: Nessage to include in Pfain tex When using HTTP protocol this request. This parameter allows using yo	e notification data. • text is used for QUERY_STRING in GET #iables
Send Port	0	O Port Number:	

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Art	Identifiant Skyresponse
CAMERA_UNREACHABLE_DETECTION	77431
HALLWAY_WANDER_DETECTION	77432
IN_BATHROOM_DETECTION	77425
INTRUDER_IN_ROOM_DETECTION	77433
MAN_DOWN_DETECTION	77421
MISSING_BED_DETECTION	77426
OUT_OF_BED_DETECTION	77423
OUT_OF_ROOM_DETECTION	77424
PERSON_VISIBLE_DETECTION	77427
PERSON_GETTING_UP_FROM_CHAIR_DETECTION	77428
PERSON_GETTING_UP_IN_BED_DETECTION	77429
SOEB_DETECTION	77422
STAFF_ENTERING_ROOM_DETECTION	77430
NO_DETECTION	77434
IN_BED_DETECTION	77434

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MOBOTIX AG Kaiserstrasse 67722 Langmeil Germany

+49 6302 9816-0 info@mobotix.com www.mobotix.com

MOBOTIX, the MOBOTIX Logo, MxControlCenter, MxEasy, MxPEG, MxDisplay and MxActivitySensor are trademarks of MOBOTIX AG registered in the European Union, the U.S.A. and in other countries • Subject to change without notice • MOBOTIX do not assume any liability for technical or editorial errors or omissions contained herein • All rights reserved • © MOBOTIX AG