Sensore intelligente MOBOTIX NurseAssist alimentato da Kepler Vison Technologies

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	Audio Messages	Access Control Settings	Analytics Events	Alarms and Events	Generic Events	<
Analytics ev	ents					
I Enabled	ł					
Port						
9090						
Security						
Events allo	wed from:					
All netv	work addresses					
	ed network addresse	es:				
A	ddress					
	Import					
	Import					
	Import					
	Import					
	Import					
	Import					
	Import					
	Import					
	Import					
	Import					

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#### Windows Security

- <-≡
  - 命 Home
  - ♡ Virus & threat protection
  - Account protection
  - <sup>())</sup> Firewall & network protection
  - 므 Device security
  - $\widehat{\otimes}$  Device performance & health
- A Family options

#### $\bigcirc$ Virus & threat protection

Protection for your device against threats.

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Profiles & Options	Value	Explanation
IP Notify Type	Custom Configuration	Profefined Configuration:     Profefined Configuration:     MrCC Alarm Transfer production network messages to the NACC alarm list. Acknowledge Reputed/prompts the     NACC user to confirm the message. If the alarm is not acknowledged within the specified acknowledge time, th     camera triggers a transmission error.     Select Castion Configuration to see the extended configuration.
Destination Address	192.168.10.20.8000	Destination Addresses: Receiver IP address and port. Separate IP address and port using a colon. Enter one address, per line.
	Parallel send to all	Send Order: Send notification to one or more destinations. Sequential and parallel will send a motification to each destination address. Send to nee or ever will stop also the first successful is definition or will try the next address if unsuccessful.
Data Protocol	Raw TCP/IP	Transfer Protocol: Transfer notification data using these protocol headers.
	-next image-	Separation for Rew TCP/09: Enter the separator for splitting several notification parts in Raw TCP/0Pmode. Enter a unique string.
		CEX.Path Adoches CE path beginning with 1/: This parameter allows using <u>variables</u> .
		HTTP Authentication: User name and password for HTTP authentication separated by colon. Example administration
Data Type	Plain text	8 Notification Data Select type of IP notification data.
	("name":"\$(ID.NAM)","event":"\$(FPR.ENO)","eventName":"VM2","ip";"\$(ID.ETHERNET)"}	Message Mensage to include in Plain test-notification data. When using HTTP protocol this test is used for QUERY_STRING in GET request. This parameter allows using <u>variables</u> .

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Θ	c71 mx10-32-2	4-170 Action Group Details (0)
General Settings	Value	Explanation
Action Group	IQM VM2 Falling	Mame: The name is purely informational,
	Enabled	Arming:     Controls this action group:     Enabled activate the group.     Off deactivates the group.     Off deactivates the group.     Si group armed by signal input.     CS: group armed by caston signal as defined in <u>General Event Settings</u> .
	(No time table)	<ul> <li>Time Table: Time table for this action profile (Time Tables).</li> </ul>
Event Selection	Message: KeplerNurseAssist (Message: Fall_Detected) (Message: Not_in_Bed) (Message: n_Bed) (Message: Situation_Normal)	Sent Statection:     Select the events which will trigger the actions below.     Use [Cat]. Click to safect more than one event.     Events in parentheses need to be <u>activated</u> first.
Action Details	5	<ul> <li>Action Deadline: Time to wait (03600 s) before a new action can take place.</li> </ul>
	Simultaneously	Action Chaining:     Choose how the status of each subaction influences the execution of all others.     Simultaneously will actions are executed simultaneously.     Simultaneously wind first success. Simultaneously.     Simultaneously and first success. Simultaneously.     Consecutively will actions are executed in the specified order.     Consecutively will actions are executed in the specified order.     Consecutively will action success. Consecutive execution, but as soon as one action succeeds; the following actions     are not executed.     Consecutively will first failure. Consecutive execution, but as soon as one action succeeds; the following actions     are not executed.     Consecutively will first failure.     Consecutively will first failure.     Consecutively will first failure.     Consecutively and first failure.     Consecutively
Actions	Value	Explanation
Action 1 Delete	IP Notify: IQM VM2 0	Action Type and Profile:     Select the Action Profile to be executed.      Action Timeout or Duration:     If this action provide insert that the insertified (0, 3600 k) it is aborted and returns an error: B to disarbane.

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Profiles & Options	Value	Explanation
IP Notify Type	нттр/нттрs	Predefined Configuration: "MxCC Alarm"sends predefined network messages to the MxCC alarm list. Acknowledge Required prompts the MxCC user to confirm the message. If the alarm is not acknowledged within the specified acknowledge time, the camera triggers a transmission error. Select Custom Configuration to see the extended configuration.
Destination Address	dev-smartsense.ascom.com	Destination Addresses: Receiver IP address and port. Separate IP address and port using a colon. ✓ Enter one address per line.
HTTP Method	GET	HTTP Nethod:     Transfer data using one of these HTTP methods.
	/api/thirdparty/alerts/trigger-alert?SensorId=c71DEYM&Alert	Nar CGI-Path: Absolute CGI path beginning with '/'. This parameter allows using <u>variables</u> .
Security	HTTPS	HTTP Schema:     Transfer data using these HTTP schemas.
	Basic authentication	Authentication Method: Method for HTTP Authentication.
		HTTP Authentication: User name and password for HTTP authentication separated by colon. Example: adminumeinsm
Data Type	NOSL	Notification Data:     Select type of IP notification data.

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/api/thirdparty/clients/status?SensorId={{SensorId}}&SensorTy
atusName={{StatusName}}

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Profiles & Options	Value		Explanation
IP Notify Type	нттр/нттрs	٠	Predefined Configuration: MACC June Tests predefined network messages to the kaCC alarm list, Acknowledge Required prompts MACC user to confirm the message, if the alarm is not acknowledged within the specified acknowledge tim the camera triggers a transmission emat, Select Caster Configuration to see the etended configuration.
Destination Address	api.demo.connected-health.nl:3000		Destination Addresses: Receiver IP address and port. Separate IP address and port using a colon. Enter one address per line.
	Parallel send to all	٠	Send Order: Send notification to one or more destinations. Sequentiariand parallel will send a notification to <b>each</b> destination address. Send to next on enrorwill stop after the <b>first</b> successful notification or will try the next address if unsucces
HTTP Method	POST	٠	HTTP Method: Transfer data using one of these HTTP methods.
	/kepler/event		CGI-Puth: Absolute CGI path beginning with 'Y. This parameter allows using <u>variables</u> .
Security	HTTP/1.1	٥	HTTP Schema: Transfer data using these HTTP schemas.
	Basic authentication	٥	Authentication Method: Method for HTTP Authentication.
		۵	HTTP Authentication: User name and password for HTTP authentication separated by colon. Example: adminumeinsm
Data Type	JSON	٠	Notification Data: Select type of IP notification data.
	{     "uuid": \$(MSG.LOCAL.KeplerNurseAssist.notification.uuid),     "timestamp": \$(MSG.LOCAL.KeplerNurseAssist.notification.timestamp),     "type"; \$(MSG.LOCAL.KeplerNurseAssist.notification.notificationType),     "stream": \$(DDAMM);     "message": \$(MSG.LOCAL.KeplerNurseAssist.notification.notificationType) }		Message to Message to include in Plain test notification data. When using HTTP protocol this test is used for QUERY_STRING in GET request. This parameter allows using <u>satisfields</u> .
Send Port	0	6	Port Number:

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General Settings	Value	Explanation
Action Group	FallDetected	Mame: The name is purely informational.
	Enabled	Arming:     Controls this action group:     Enabled, activate the group.     Off deactivate the group.     St. group armed by signal input.     CS: group armed by custom signal as defined in <u>General Event Settings</u> .
	(No time table)	Time Table: Time table for this action profile ( <u>Time Tables</u> ).
Event Selection	(Image Analysis: AS) (Image Analysis: VM) (Image Analysis: VM2) Message: NotInBed Message: FallDetected	Event Selection: Select the events which will origger the actions below. Use [Crt]-Click to select more than one event. Events in parentheses need to be <u>activated</u> first.
Action Details	5	Action Deadtime:     Time to wait (0500 s) before a new action can take place.
	Simultaneously	Action Chaining:     Action Chaining:     Simultaneously. All actions are executed simultaneously.     Simultaneously and first success Simultaneously.     Simultaneously and first success Simultaneous execution, but as soon as one action succeeds [i.e. has been     completed or the phone is picked up, all others are terminated.     Consecutively. All actions are executed in the specified order.     Consecutively undiffers success. Consecutive execution, but as soon as one action succeeds, the following     actions are not executed.     Consecutively undiffers follow: Consecutive execution, but as soon as one action fails, the following actions are     not executed.
Actions	Value	Explanation
Action 1 Delete	IP Notify: Connected Health	Action Type and Profile:     Select the Action Profile to be executed.      Action Timeout or Duration:     If this action runs longer than the time specified [03600 s], it is aborted and returns an error; 0 to deactivate.     For Image Profile action this is the duration and no error returns.

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Ð	c71 Mobotix-c71-KVEC	P Notify Profiles	0 0 = 1
IP Notify Profile 10 S	kyresponseFallDetected Delete		2
Profiles & Options	Value	Expla	nation
P Notify Type	HTTP/HTTPS	Predef     MorCC.     list.Act     messag     acknow     Select	ined Configuration: Alarm "sends predefined network messages to the MACC alarm inowledge Required prompts the MACC user to confirm the e.g. If the alarm is not acknowledged within the specified dedge time, the camera triggers a transmission error. Custom Configuration to see the extended configuration.
Destination Address	sandbox.skyresponse.com	Destin Receive Separa A Enter o	stion Addresses: r IP address and port. te IP address and port using a colon. ne address per line.
	Parallel send to all	<ul> <li>Send 0 Send n Sequen address Send to try the</li> </ul>	voler: otification to one or more destinations. trial and parallel will send a notification to <b>each</b> destination s. nent on error will stop after the <b>first</b> successful notification or w next address if unsuccessful.
ITTP Method	POST	HTTP N     Transfe	fethod: r data using one of these HTTP methods.
	/api/alarms	CGI-Pa Absolut This pa	th: te CGI path beginning with '/'. rameter allows using <u>variables</u> .
iecurity	HTTPS	HTTP S     Transfe	ichema: r data using these HTTP schemas.
	None	Authen     Method	tication Method: for HTTP Authentication.
		Liser na	wthentication: me and password for HTTP authentication separated by colon. le: admin:meinsm
Data Type	JSON	Notific     Select t	ation Data: ype of IP notification data.
	{ "authentication": { "identifier": " "secret": "39yAaWeyfwPJWMjg", "product": "KPLR"	Messag Messag When u request This pa	pe: e to include in Pflain text notification data. sing HTTP protocol this text is used for QUERY_STRING in GET L rameter allows using <u>variables</u> .
iend Port	0	O Port N	amber: measure from this summer part (5 for automatic)

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Art	Identificatore Skyresponse
CAMERA_UNREACHABLE_DETECTION	77431
HALLWAY_WANDER_DETECTION	77432
IN_BATHROOM_DETECTION	77425
INTRUDER_IN_ROOM_DETECTION	77433
MAN_DOWN_DETECTION	77421
MISSING_BED_DETECTION	77426
OUT_OF_BED_DETECTION	77423
OUT_OF_ROOM_DETECTION	77424
PERSON_VISIBLE_DETECTION	77427
PERSON_GETTING_UP_FROM_CHAIR_DETECTION	77428
PERSON_GETTING_UP_IN_BED_DETECTION	77429
SOEB_DETECTION	77422
STAFF_ENTERING_ROOM_DETECTION	77430
NO_DETECTION	77434
IN_BED_DETECTION	77434

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MOBOTIX AG Kaiserstrasse 67722 Langmeil Germany

+49 6302 9816-0 info@mobotix.com www.mobotix.com

MOBOTIX, the MOBOTIX Logo, MxControlCenter, MxEasy, MxPEG, MxDisplay and MxActivitySensor are trademarks of MOBOTIX AG registered in the European Union, the U.S.A. and in other countries • Subject to change without notice • MOBOTIX do not assume any liability for technical or editorial errors or omissions contained herein • All rights reserved • © MOBOTIX AG